

IV INFUSION SET INSTRUCTIONS FOR USE



RELATED PRODUCTS

IV infusion sets with needle-free/needle-type Y port, with or without UV protection

PRODUCT DESCRIPTION

ONCOERA Infusion Sets are used for safe intravenous drug and fluid transfer to the patients in all wards where IV drug administration is carried out meticulously.

INSTRUCTIONS FOR USE

1. Close the clamps of infusion set.
2. Open the protective cap of spike tip. Plunge the spike into the port of liquid/drug to be used in one stroke.
3. Open the clamps. Allow the drip chamber fill halfway with liquid and allow the liquid move through the set.
4. Remove the air inside the set by opening all the clamps and by applying pressure to the bag containing the liquid. Close all clamps.
5. Connect the distal end of infusion set to the patient's IV device.
6. Check the patient's vascular access by opening all the clamps on the infusion set.
7. Adjust the flow rate of the liquid with the roller clamp.
8. Perform the drug infusion.
9. Close the clamps of the infusion set when the infusion is finished.
10. If different drugs will be administered during or after the administration, inject the relevant drug into the infusion line through the needle/needle-free Y port on the set.
11. Close all clamps of the infusion set when the procedures are completed. Dispose of the set as a whole.
12. **IMPORTANT NOTE:** Silicone line optimizes the sensitivity of the administration set to deliver fluid through the infusion pump. Gravity infusion sets do not have a silicone line.

WARNINGS & PRECAUTIONS

1. It should be used by healthcare personnel.
2. **NOT SUITABLE** for blood products containing cells.
3. **DO NOT USE** if the package has been opened or damaged.
4. **DO NOT USE** expired products.
5. Check that all protective caps are present before use.
6. **DO NOT ACCESS** needle-free valves with a needle or blunt cannula.
7. **DO NOT USE** in patients who are hypersensitive to any of the materials used.
8. If the silicone part of the valve becomes damaged or does not return to its original form after multiple access, replace the set. This may cause leakage.
9. **DO NOT OVERTIGHTEN** the connections.
10. If it is leaking, replace the product with a new one.
11. Replace the product with a new one in case the set separates from the bonding areas.
12. The product is for single use only. **DO NOT RESTERILIZE.**
13. **DO NOT LEAVE** infusion sets unattended.
14. If you observe that a drug or liquid has damaged the infusion set, replace the set with a new one and inform our company by naming the drug.
15. When the drug administration process is finished, dispose of the infusion set and the connected transfer sets as a whole without separating them.



IV INFUSION SET INSTRUCTIONS FOR USE

STORAGE TEMPERATURE

Product storage temperature is 15 – 30°C.

TIME OF USE

Product usage time is 48 hours.



















SHELF LIFE

The shelf life of the product is 5 years.

USERS

Specialist physicians, nurses, pharmacists and other relevant health-trained workers.

LABEL DISCLOSURES

 Manufacturer Information	 Notified Body Number	 Manufacturing Date	 Expiry Date	 Lot Number
 Reference Number	 Sterilized with EO Gas	 Do Not Re-sterilize	 Do Not Use If Package is Damaged	 Keep Away From The Sun
 See User Manual	 Storage Temperature Limits	 Single Use Only	 Caution	 Pyrogen Free
 DEHP – Free	 Latex – Free	 Medical Device		